# Addresses

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## Table of Contents

<b>1. Introduction</b> .....	<b>35</b>	<b>8. Feeding in Continuous Mode</b> .....	<b>46</b>
1.1 Intended Use .....	35	<b>9. Feeding in Interval Mode</b> .....	<b>47</b>
1.2 Support .....	35	<b>10. Pausing a Nutritional Therapy</b> .....	<b>49</b>
<b>2. Safety Information</b> .....	<b>35</b>	<b>11. Delivery Information</b> .....	<b>50</b>
2.1 Warnings and Precautions .....	35	11.1 View/Clear Delivery Information in Continuous Mode .....	50
<b>3. System Overview</b> .....	<b>36</b>	11.2 Rolling 30-day Volume History .....	50
3.1 Pump and Accessory .....	3, 4, 36	11.3 View/Clear Delivery Information in Interval Mode .....	51
3.2 Administration Set .....	4, 36	<b>12. User Settings</b> .....	<b>52</b>
3.3 Button Function .....	37	12.1 Alarm level, Backlight, Autodim, End of Nutritional Therapy Alarm .....	52
3.4 Display .....	37	12.2 Changing the Language .....	53
3.5 Battery .....	39	<b>13. Locking and Unlocking the Keypad</b> .....	<b>53</b>
<b>4. Pump Mode Overview (Continuous vs. Interval)</b> .....	<b>40</b>	<b>14. Advanced Settings</b> .....	<b>54</b>
4.1 Continuous Mode: Menu Overview .....	41	<b>15. Alarms &amp; Troubleshooting</b> .....	<b>55</b>
4.2 Interval Mode: Menu Overview .....	41	15.1 Alarm Handling .....	55
<b>5. Clear Nutritional Therapy Settings and Delivery Information</b> .....	<b>42</b>	15.2 Alarm Overview and Troubleshooting .....	56
<b>6. Pump Setup</b> .....	<b>42</b>	<b>16. Cleaning &amp; Disinfection</b> .....	<b>58</b>
6.1 Connect to the AC power adapter .....	42	<b>17. Maintenance</b> .....	<b>59</b>
6.2 Use with a pole clamp .....	42	<b>18. Accessories</b> .....	<b>59</b>
6.3 Power On and Off .....	43	<b>19. Limited Warranty</b> .....	<b>60</b>
6.4 Inserting the Administration Set .....	43	<b>20. List of Symbols</b> .....	<b>60</b>
6.5 Filling the Administration Set (Priming) .....	44	<b>21. Specifications</b> .....	<b>61</b>
<b>7. Quick Start Guides</b> .....	<b>45</b>	<b>22. Recycling and Disposal</b> .....	<b>62</b>
7.1 Continuous Mode Feeding .....	45		
7.2 Interval Mode Feeding .....	45		

## 1. Introduction

Compat Ella® is a small, lightweight and portable rotary peristaltic pump used to deliver programmed amounts of enteral tube feed at selectable delivery rates to the gastrointestinal system. The pump should be used only after proper training regarding care, use and maintenance of the Compat Ella® pump and accessories.

### 1.1 Intended Use

The Compat Ella® enteral feeding pump is intended to be used for the delivery of enteral tube feed and hydration at controlled rates to the gastrointestinal system. It is intended for use with adult and pediatric patients requiring enteral feeding and hydration in hospital, nursing home and home care environments.

### 1.2 Support

If you need help with the Compat Ella® pump, please contact your Nestlé Health Science representative or go to: [www.nestlehealthscience.com](http://www.nestlehealthscience.com).

## 2. Safety Information

Read the entire Manual before operating the Compat Ella® pump.

Read the entire administration set IFU (Instructions for Use) before installing an administration set in the Compat Ella® pump.

**Warning:** Describes a hazard or unsafe practice that if not avoided can result in severe bodily injury, death or property damage.

**Precaution:** Describes a hazard or unsafe practice that if not avoided can result in minor bodily injury or mainly property damage.

*Note: Describes something that is more important than the rest of the text.*

### 2.1 Warnings and Precautions

#### **Warnings:**

- Not for intravenous use. For enteral use only.
- If pump is damaged or altered, do not use it.
- Make sure power cord plug is completely dry before and during use.
- Do not attempt to attach the “country specific” AC adapter to AC power without it attached to the AC adapter body.
- Do not use in the presence of flammable anesthetics or other flammable gases (including enriched oxygen environments), near flammable fluids or objects, or in the presence of oxidizing agents.
- During pump start up, confirm all the screen segments are visible. If any are missing or if the screen is not readable, do not use the pump.
- When turning the pump on, if you do not hear the tones, do not use the pump.
- Use only prescribed enteral tube feed supplied by a qualified healthcare professional and hydration fluids such as water, tea.
- Use only Compat Ella® administration sets.
- Inspect the Compat Ella® administration set. Do not use if damaged or altered.
- Always ensure appropriate, timely and regular patient monitoring, especially if an interruption of the tube feeding delivery may lead to a patient hazard.

- Replace the Compat Ella® administration set every 24 hours to maintain delivery accuracy and help prevent the growth of potential harmful contamination.
- Give particular attention to the risk of strangulation with cables and sets.
- Do not program Rate and Volume combinations that exceed 24 hours.
- Do not submerge the pump in water.
- Do not use in a dishwasher.
- To avoid electrical shock, never clean while the pump is on or plugged in.
- Do not autoclave.
- The use of harsh cleaners/disinfectants and isopropyl alcohol, may damage the pump housing.

#### Precautions:

- Do not use in strong magnetic fields. e.g. MRI (Magnetic Resonance Imaging) environments.
- Ensure that the administration set is not connected to a patient when priming.
- Verify that the administration set is correctly installed, is connected to the nutrition container and is ready to prime.
- When programming a new therapy, make sure to clear the **VOLUME GIVEN THIS THERAPY** parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).
- The battery capacity is an estimate. If you are unsure that enough capacity remains for your intended use, connect to the AC power until recharged.
- The advanced settings should only be adjusted by a qualified healthcare professional.

## 3. System Overview

### 3.1 Pump and Accessory (see pages 3-4)

- |                                |                                     |   |
|--------------------------------|-------------------------------------|---|
| <b>A</b> Pump door             | <b>G</b> Upstream pressure sensor   | <b>M</b> Rear pump label with Quick Start Guide |
| <b>B</b> Door latch            | <b>H</b> Downstream pressure sensor | <b>N</b> Loudspeaker                            |
| <b>C</b> Pump rotor            | <b>I</b> Air sensor                 | <b>O</b> Serial number label                    |
| <b>D</b> Keypad                | <b>J</b> Rubber foot pad            | <b>P</b> Legal manufacturer label               |
| <b>E</b> AC power status light | <b>K</b> AC power connection        | <b>Q</b> AC power adapter                       |
| <b>F</b> Color display         | <b>L</b> Pole clamp fitting         | <b>R</b> Pole clamp                             |

### 3.2 Administration Set (see page 4)

- |                           |  |
|---------------------------|--|
| <b>S</b> ENPlus connector | <b>U</b> Medication port (shown : ENFit™ + Luer/Oral/ENLock) |
| <b>T</b> Cassette         | <b>V</b> Distal end (shown : ENFit™ + ENLock)                |

*Note: Different versions of the administration sets are available. To obtain further information please contact your Nestlé Health Science representative or go to [www.nestlehealthscience.com](http://www.nestlehealthscience.com).*

### 3.3 Button Function

The pump always retains in memory the on-screen settings and values when scrolling from one screen to the next, as well as when the pump is turned off.

Button	Primary Function	Secondary Function
	Turn pump on or off (hold 2 seconds (2"))	
	Start / Pause pump	Snooze alarm
	Auto-prime (hold 2 seconds (2")) Press again to stop auto-prime early	
	Scroll through rate, volume and interval programming	Switch between continuous and Interval modes (hold 5 seconds (5"))
	Decrease and increase values (hold: accelerates speed)	Lock and unlock keypad (press both buttons together)
	Clear the on-screen value to default or 0	Reset all program values to default and clear recent feed history (hold button 3 seconds (3"))
	Scroll through recent feed history	Access last rolling 30-day Volume History (hold button 2 seconds (2"))
	Scroll through pump settings	Change language [See section 12.2]

 **Precaution:** Ensure that the administration set is not connected to a patient when priming.

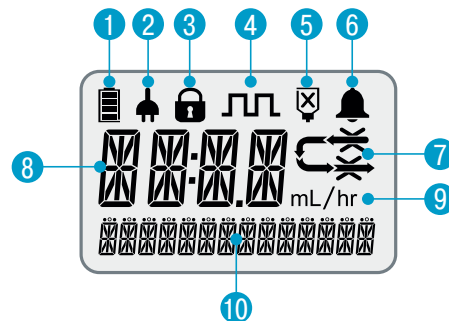
### 3.4 Display

 **Warning:** During pump start up, confirm all the screen segments shown in the diagram hereafter are visible. If any are missing or if the screen is not readable, do not use the pump.

#### Screen Layout

The Compat Ella® pump features a colour display with alphanumeric characters, symbols and a backlight.

- 1 Battery charge level
- 2 Plugged into AC power
- 3 Keypad locked
- 4 Interval mode activated
- 5 Tube feed container empty / Air In Line
- 6 Alarm indicator
- 7 Run icon with occlusion indicators: the arrow segments light sequentially when the pump is running.
- 8 Main info line: displays alpha numerical values for programming and alarms.
- 9 Main info label: mL, hr, or mL/hr.
- 10 Text message line: provides miscellaneous info on pump status, programming and alarms.

*Note: The backlight of the display turns off 10 seconds after the last key is pressed if the pump is disconnected from AC power and the Autodim feature is activated.*

### Colour Screen Overview

The pump screen uses different colours to help distinguish pump status and on-screen information.

Programming information:  
**black** characters

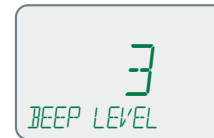
Normal operating conditions:  
**green** icons with **black** characters

Alarms:  
**red** characters and icons

Information:  
**blue** characters

Settings:  
**green** characters

*Note: When the backlight is dimmed, all on-screen information is shown in black.*

### Default / Run Screen Overview

When the pump is running normally, the following screen will be shown:

1. The sequentially-appearing green arrows  show that the pump is currently running a programmed nutritional therapy.
2. The 120 mL/hr shows the rate the pump is running in mL/hr.
3. The 100/500 mL shows that 100 mL has been given out of the 500 mL programmed volume. If /CONT is displayed, this means no volume limit has been set and the pump will run continuously until the tube feed runs out.
4. The battery is fully charged and is plugged into the AC power.

*Note: Compat Ella® only pumps fluid a little at a time. Normal function is to pump fluid for a while, then stop pumping fluid and then start pumping fluid again. This means that you will not hear the motor running all the time. The sequentially-appearing green arrows  are continually shown, which means that the pump is functioning correctly.*

## 3.5 Battery

First Use: Plug in the pump and fully charge the battery for a minimum of 6 hours before using the pump on battery power only. This will maximize the charge and life of the battery.

Battery capacity: 24 hours at 125 mL/hr.

Recharge: approximately 6 hours from fully depleted to fully charged.

### Battery Charge Indicators

Battery Fully Charged

Battery Partially Charged

Pump on External Power

Battery Charging

**Precaution:** The battery capacity is an estimate. If you are unsure that enough capacity remains for your intended use, connect to the AC power until recharged.

## 4. Pump Mode Overview (Continuous vs. Interval)

The Compat Ella® portable enteral feeding pump has two operational modes: Continuous and Interval.

### Continuous Mode

Continuous mode is used to deliver tube feed at defined rates, with or without a volume limit.

### Interval Mode

Interval mode allows a defined bolus to be delivered multiple times during the day at programmed intervals, with or without a total volume limit. An Interval is the time from the start of one bolus to the start of the next bolus.

### Switch Between Continuous and Interval Modes

Button	Direction/Result
 <p>5"</p>	<p>To change between operating modes, press and hold  for 5 seconds (5").</p> <p>A specific icon shows that Interval mode is activated.</p> <p>A message will appear confirming activation or de-activation.</p> <div data-bbox="1321 456 1493 602" style="border: 1px solid gray; padding: 5px; margin-top: 10px;"> <p style="text-align: center;">≧⏏≦</p> <p style="text-align: center;">INTERVAL MODE</p> <p style="text-align: center;">ACTIVATED</p> </div>

*Note: A qualified healthcare professional must enable the use of the interval mode. (See advanced settings)*

## 4.1 Continuous Mode: Menu Overview

Press  to scroll through *SET RATE* and *SET VOLUME*.

Pressing  again cycles back to *SET RATE*

Button	Device Display	Function
	<i>SET RATE</i>	The delivery speed in mL/hr at which the pump will run. <ul style="list-style-type: none"> <li>• Rate may be set from 1-600 mL/hr in 1 mL/hr increments.</li> <li>• Default value 0 mL/hr</li> </ul>
	<i>SET VOLUME</i>	The volume of tube feed to be given, in mL. <ul style="list-style-type: none"> <li>• Volume may be set from 1-4000 mL in 1 mL increments.</li> <li>• Default value: <i>CONT</i>*</li> </ul>
	<i>SET RATE</i>	Return to <i>SET RATE</i> screen

### Note:

- Both *RATE* and *VOLUME* must be set/ reviewed if either one is changed or the pump has just been turned on. Only then can the pump be started.

\* Continuous: no feeding volume has been set. The pump will run continuously until the formula runs out.

## 4.2 Interval Mode: Menu Overview

Press  to scroll through *SET RATE* and *SET BOLUS*,

*SET INTERVAL* and *SET TOTAL VOLUME*. Pressing  again cycles back to *SET RATE*

Button	Device Display	Function
	<i>SET RATE</i>	The delivery speed in mL/hr at which the pump will run. <ul style="list-style-type: none"> <li>• Rate may be set from 1-600 mL/hr in 1 mL/hr increments.</li> <li>• Default: 0 mL/hr</li> </ul>
	<i>SET BOLUS</i>	The bolus volume of tube feed to be given before each interval pause, in mL. <ul style="list-style-type: none"> <li>• Bolus volumes may be set from 1-2000 mL in 1mL increments.</li> <li>• Default: 0 mL</li> </ul>
	<i>SET INTERVAL</i>	The interval between start of one bolus and start of the next bolus. <ul style="list-style-type: none"> <li>• Interval may be set from 00:15-12:00 hr in 15-minute increments. Automatically calculated when Rate and Bolus have been programmed.</li> <li>• Default: 00:15 hr</li> </ul>
	<i>SET TOTAL VOLUME</i>	The total volume in mL to be given during the entire nutritional therapy. <ul style="list-style-type: none"> <li>• Total volume may be set from 1-4000 mL in 1mL increments.</li> <li>• Default value: <i>CONT</i>*</li> </ul>
	<i>SET RATE</i>	Return to <i>SET RATE</i> screen

*Note: RATE, BOLUS, INTERVAL and TOTAL VOLUME must all be set / reviewed if either one is changed or the pump has just been turned on. Only then can the nutritional therapy be started.*

## 5. Clear Nutritional Therapy Settings and Delivery Information

Button	Direction / Result
	<p>Hold down  for 3 seconds (3") to clear all nutritional therapy settings.</p> <ul style="list-style-type: none"> <li>• Nutritional therapy values cleared to default: <i>RATE, VOLUME, BOLUS, INTERVAL, TOTAL VOLUME</i></li> <li>• Volume history values cleared to default: <i>VOLUME GIVEN THIS THERAPY, CUMULATIVE VOLUME</i></li> <li>• The pump displays a <i>THERAPY RESET</i> confirmation message.</li> </ul>

*Note: This does not affect any other pump settings (such as alarm level, backlight etc.), language or rolling 30-day volume history.*

## 6. Pump Setup

### 6.1 Connect to the AC power adapter

See pump and accessories overview (page 4)

The AC power adapter is delivered with multiple country-specific adapters. To change the country specific adapter, press the button on the AC power adapter and rotate country specific adapter to release it. Install the new country specific adapter and rotate until you hear a click.

**Warning:** Do not attempt to attach the “country specific” AC adapter to AC power without it attached to the AC power adapter body.

### 6.2 Use with a pole clamp

See pump and accessories overview (page 4)

### 6.3 Power On and Off

Button	Direction / Result
	<p><b>Power On</b></p> <p>Press and hold  for 2 seconds (2") to power on. You should hear 3 tones. The pump will perform a diagnostic of all systems and will display all screen segments briefly as well as the time. Ensure that all the segments are working and that the time is correct.</p>
	<p><b>Power Off</b></p> <p>Press and hold  2 seconds (2") to power off. You should hear 3 tones as the pump shuts down.</p> <p><i>Note: All nutritional therapy parameters, volume history and user settings are retained in the pump's memory for re-use when the pump is next turned on.</i></p>

 **Warning:** When turning the pump on, if you do not hear the tones or do not see all the screen segments, do not use the pump. Contact your Nestlé Health Science representative or Nestlé approved service provider.

#### Power Status Light

Solid green = Pump is plugged into the AC power.

### 6.4 Inserting the Administration Set

-  **Warnings:**
- Use only Compat Ella® administration sets.
  - Inspect the Compat Ella® administration set. Do not use if damaged or altered.
  - Replace the Compat Ella® administration set every 24 hours to maintain delivery accuracy and help prevent the growth of potential harmful contamination.

#### Loading the Cassette

See sequence on page 2 to load set into the pump.

Fig. 1: open door by lifting the thumb tab

Fig. 2, 3: wrap cassette loop around rotor. Note the arrow which shows correct orientation

Fig. 4, 5: using your thumb, slide the cassette to the right to clip the cassette in the pump

Fig. 6: close the door by pressing on the "PUSH" symbol.

*Note: Compat Ella® Administration Sets are equipped with a patented Auto-Anti-Free-Flow (AAFF) mechanism, which is always active when needed. The AAFF mechanism is only released when the cassette is inserted in the pump and the door is closed.*

## 6.5 Filling the Administration Set (Priming)

Priming fills the administration set with tube feed.

### Precautions:

- Before priming, always verify that the administration set is not connected to the patient.
- Verify that the administration set is correctly installed, is connected to the nutrition container and is ready to prime.

### Auto-prime

Auto-prime should always be used if possible.

Button	Direction/Result
	<p>Press and hold  for 2 seconds (2") to prime automatically. While priming, the pump displays the run icon together with a <i>PRIMING</i> message. You may stop auto-prime at any time by pressing  again.</p>

### Notes:

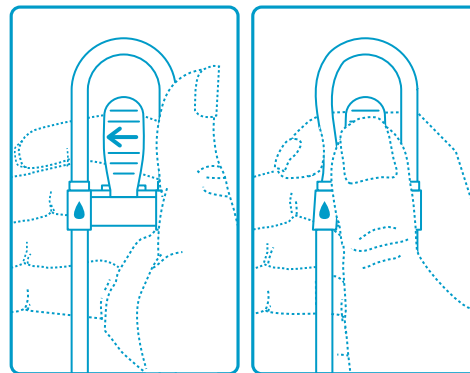
- The pump is programmed to automatically deliver a pre-set volume of tube feeding to prime the tubing.
- Once the pre-set volume of tube feeding has been given, the pump stops automatically and displays *END OF PRIMING*.
- When prime is complete, the tubing should be checked to ensure that the tubing is completely filled. If additional priming is required, the auto-prime can be repeated.
- The air detection alarm is disabled during priming.

### Manual Prime

If the auto-prime function is unwanted, manual priming can be performed. Gently pinch the tubing on the cassette as shown and hold this position until all air is removed from tubing. If needed, gently squeeze and tilt container at the same time to assist fluid flow. If fluid does not flow, the pinch pressure may be too strong.

### Note:

- It is important to only gently pinch the tubing above the symbol  to avoid damaging the Auto-Anti-Free-Flow mechanism located underneath. Use only the flat, soft part of your finger. Do not use fingernails as this may puncture the tubing.
- High viscosity feeds may not flow effectively using the manual priming method and should be primed using the pump auto-prime function

## 7. Quick Start Guides

### 7.1 Continuous Mode Feeding

Step	Button	Function
1	 2"	Turn the pump on
2		Review the <i>VOLUME GIVEN THIS THERAPY</i>
3		Clear the <i>VOLUME GIVEN THIS THERAPY</i>
4	 2"	Prime the administration set
5		Use <b>+</b> and <b>-</b> to <i>SET RATE</i> 1 – 600 mL/Hr
6		Use <b>+</b> and <b>-</b> to <i>SET VOLUME</i> 1 – 4000 mL/CONT*
7		Start the nutritional therapy

\* Continuous: no feeding volume has been set. The pump will run continuously until the formula runs out.

#### Precautions:

- When programming a new therapy, make sure to clear the *VOLUME GIVEN THIS THERAPY* parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

### 7.2 Interval Mode Feeding

Step	Button	Function
1	 2"	Turn the pump on
2		Review the <i>VOLUME GIVEN THIS THERAPY</i>
3		Clear the <i>VOLUME GIVEN THIS THERAPY</i>
4	 2"	Prime the administration set
5		Use <b>+</b> and <b>-</b> to <i>SET RATE</i> 1 – 600 mL/Hr
6		Use <b>+</b> and <b>-</b> to <i>SET BOLUS</i> 1 – 2000 mL
7		Use <b>+</b> and <b>-</b> to <i>SET INTERVAL</i> 0:15 hr – 12:00 hrs
8		Use <b>+</b> and <b>-</b> to <i>SET TOTAL VOLUME</i> to be given 1 – 4000 mL/CONT*
9		Start the nutritional therapy

## 8. Feeding in Continuous Mode

Continuous mode is used to deliver tube feed at defined rates, with or without a volume limit.

Step	Button	Function	Instruction
1	 2"	Turn the pump on	The most recent nutritional therapy settings are always saved in memory. When power is turned on, the display shows the last programmed flow rate and volume and the actual volume given.
<b>Load the administration set in the pump</b>			
2		Review the volume given this therapy	Press  and review the <i>VOLUME GIVEN THIS THERAPY</i> .
3		Clear the volume given this therapy	If desired, press  to clear the <i>VOLUME GIVEN THIS THERAPY</i> . <i>Note: This should always be done when starting a new nutritional therapy.</i>
4	 2"	Prime the administration set	<ul style="list-style-type: none"> <li>• Hold 2 seconds (2") to activate auto-prime. The pump will fill the administration set with a predetermined volume.</li> <li>• Press again to stop priming at any time.</li> </ul> <i>Note: Do not prime when connected to the patient's feeding tube.</i>
5		Set the rate	Press  to access the <i>SET RATE</i> screen and use  and  to define the rate of tube feed delivery. <ul style="list-style-type: none"> <li>• The rate may be set from 1-600 mL/hr in 1 mL/hr increments.</li> </ul> <i>Note: Press  to reset value.</i>
6		Set the volume	Press  to scroll to the <i>SET VOLUME</i> screen and use  and  to define the amount of tube feed to be delivered. <ul style="list-style-type: none"> <li>• Feeding with a dose limit: The value may be set from 1-4000 mL in 1 mL increments.</li> <li>• Continuous feeding without volume limit: Set the value to CONT (Continuous), the pump will run continuously until the nutrition container is empty.</li> </ul> <i>Note: Press  to reset value.</i>

Ensure nutritional therapy settings correspond to the nutritional therapy prescribed.

Connect the administration set to the patient's feeding tube.

7		Start the nutritional therapy	Press  to run the nutritional therapy. The pump will automatically run the nutritional therapy and display the progress.
---	---	-------------------------------	---

#### Precautions:

- When programming a new therapy, make sure to clear the *VOLUME GIVEN THIS THERAPY* parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

#### Notes:

- Holding  or  changes values rapidly:
  - When setting rate, the value will pause at 1, 50, 125, 250, 600 mL/hr.
  - Total Volume will also pause at 1, 125, 250, 500, 750, 1000, 1250, 1500, 2000 and 4000 mL.
- When at maximum value, pressing  again will go to the minimum value and vice versa e.g. rate from 600 mL/hr to 1 mL/hr.
- The pump will sound the alarm when a programmed volume is completely given or the nutrition container runs out. The **END OF THERAPY** alarm may be temporarily deactivated (see 12.).
- Patients sensitive to nutrition delivery should be monitored closely at all times.

## 9. Feeding in Interval Mode

Interval mode is used to deliver multiple bolus volumes at regular intervals, at defined rates, with or without a volume limit.

Step	Button	Function	Instruction
1	 2"	Turn the pump on	The most recent nutritional therapy settings are always saved in memory. When power is turned on, the display shows the last programmed flow rate and volume and the actual volume given.
<b>Load the administration set in the pump</b>			
2		Review the volume given this therapy	Press  and review the <i>VOLUME GIVEN THIS THERAPY</i> .
3		Clear the volume given this therapy	If desired, press  to clear the <i>VOLUME GIVEN THIS THERAPY</i> . <i>Note: This should always be done when starting a new nutritional therapy.</i>

4	 2"	Prime the administration set	<ul style="list-style-type: none"> <li>• Hold 2 seconds (2") to activate auto-prime. The pump will fill the administration set with a predetermined volume.</li> <li>• Press again to stop priming at any time.</li> </ul> <p><i>Note: Do not prime when connected to the patient's feeding tube.</i></p>
5		Set the rate	<p>Press  to access the <b>SET RATE</b> screen and use  and  to define the rate of tube feed delivery.</p> <ul style="list-style-type: none"> <li>• Rate may be set from 1-600 mL/hr in 1 mL/hr increments.</li> </ul> <p><i>Note: Press  to reset value.</i></p>
6		Set the bolus volume	<p>Press  to scroll the <b>SET BOLUS</b> screen and use  and  to define the bolus volume or the amount of tube feed to be given multiple times.</p> <ul style="list-style-type: none"> <li>• Bolus may be set from 1-2000 mL in 1 mL increments</li> </ul> <p><i>Note: Press  to reset value.</i></p>
7		Set the interval	<p>Press  to scroll the <b>SET INTERVAL</b> screen and use  and  to define the time from the start of one bolus to the start of the next bolus.</p> <ul style="list-style-type: none"> <li>• Interval may be set from 00:15-12:00 hr in-15 minute increments.</li> <li>• The minimum interval possible is automatically calculated and shown, based on rate and bolus values from steps 5 and 6.</li> </ul> <p><i>Note: Press  to reset value.</i></p>
8		Set the total volume	<p>Press  to scroll the <b>SET TOT VOLUME</b> screen and use  and  to define the total amount of tube feed to be given if desired.</p> <ul style="list-style-type: none"> <li>• <b>Feeding with a Dose Limit:</b> The value may be set from 1 bolus programmed in step 6 to 4000 mL in 1 mL increments. The pump will run multiple bolus until the Total Volume limit has been reached.</li> <li>• <b>Feeding without Dose Limit:</b> Set the value to CONT (CONTINUOUS), the pump will run multiple bolus until the nutrition container is empty.</li> </ul> <p><i>Note: Press  to reset value.</i></p>

Ensure nutritional therapy settings correspond to the nutritional therapy prescribed.

Connect the administration set to the patient's feeding tube.

9		Start the Nutritional Therapy	<p>Press  to run the nutritional therapy. The pump will automatically run the nutritional therapy and display the progress.</p>
---	---	-------------------------------	--

#### Precautions:

- When programming a new therapy, make sure to clear the **VOLUME GIVEN THIS THERAPY** parameter.
- After starting a therapy, verify all therapy parameters (including rate, dose, and volume given this therapy).

#### Notes:

- Holding **+** or **-** changes values rapidly:
  - When setting rate, the value will pause at 1, 50, 125, 250, 600 mL/hr.
  - When setting Bolus, the value will pause at 1, 125, 250, 500, 750, 1000, 1250 and 1500 mL.
  - Total Volume will also pause at 1, 125, 250, 500, 750, 1000, 1250, 1500, 2000 and 4000 mL.
- When at maximum value, pressing **+** again will go to the minimum value and vice versa e.g. rate from 600 mL/hr to 1 mL/hr.
- The pump will sound the alarm when a programmed volume is completely given or the nutrition container runs out. The **END OF THERAPY** alarm may be temporarily deactivated (see 12.).
- After each bolus is complete, the pump will display an **INTERVAL PAUSE** and **TIME TO NEXT BOLUS** screen showing when the next bolus will start. This cycle will continue until the pump is turned off, a defined amount of tube feed is given or the tube feed runs out.

## 10. Pausing a Nutritional Therapy

Button	Result
	<p>At any time while the pump is running, you may pause the nutritional therapy by pressing . The Run icon disappears and the pump displays a <b>PUMP PAUSE</b> message. Press  to resume the nutritional therapy. Use the Pause mode to:</p> <ul style="list-style-type: none"> <li>• Change the feeding program</li> <li>• Change the feed container</li> <li>• Administer medication without switching the pump off</li> <li>• Silence an alarm and correct problems</li> </ul>

*Note: After 2 minutes, an alert sounds and the message **PUMP WAITING** appears in the display. Press  to snooze the alarm and to extend the pause mode by an additional 2 minutes.*

## 11. Delivery Information

### 11.1 View/Clear Delivery Information in Continuous Mode

At any time, press  to review the delivery information.

To clear : Ensure the pump is paused. Press  and then  to clear value to 0.

Button	Device Display	Function
 	VOLUME GIVEN THIS THERAPY	Display the volume in mL given for the current nutritional therapy. <i>Note: Must be manually cleared at the beginning of each new nutritional therapy. If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely if a VOLUME has been set.</i> To clear: Ensure the pump is paused. Press  and then  to clear value to 0.
	CUMULATIVE VOLUME	The cumulative volume given by the pump since the value was last cleared. Can be used to track volume history during any defined period, such as a shift or multiple nutritional therapies. To clear: Ensure the pump is paused. Press  twice and then  to clear value to 0.
		Exit to main screen.

*Note: if the volume information exceeds 9999 mL, the pump displays VOL EXCEEDS MAX. Clear value to restart counting from 0. If no button is pressed within 10 seconds, the info screens are exited.*

### 11.2 Rolling 30-day Volume History

Button	Function
  2"	Hold  for 2 seconds (2") to access the rolling 30-day volume history. Use  /  to scroll backwards and forwards by date. The value shown is the total volume given per 24-hour period, midnight to midnight for the last 30 days.

*Note: The rolling 30-day volume history cannot be cleared.*

### 11.3 View/Clear Delivery Information in Interval Mode

At any time, press  to review the delivery information and programmed nutritional therapy details. Press  to scroll through each menu item below. When the last menu is reached, pressing the  button exits the info mode. To clear: Ensure the pump is paused. Press  and then  to clear value to 0.

Button	Device Display	Function
	VOLUME GIVEN THIS THERAPY	Display the volume in mL given for the current nutritional therapy. <i>Note: Must be manually cleared at the beginning of each new nutritional therapy. If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely if a VOLUME has been set.</i> <i>To clear: Ensure the pump is paused. Press  and then  to clear value to 0.</i>
	CUMULATIVE VOLUME	The cumulative volume given by the pump since the value was last cleared. Can be used to track volume history during any defined period, such as a shift or multiple nutritional therapies. <i>To clear: Ensure the pump is paused. Press  twice and then  to clear value to 0.</i>
	TIME TO THERAPY END	The remaining time (in hours and minutes) until the end of the nutritional therapy. <i>Note: Not available if CONT has been set for TOTAL VOLUME.</i>
	TIME TO BOLUS END OR TIME TO NEXT BOLUS	<ol style="list-style-type: none"> <li>When the bolus is being given, the time left in hours and minutes before the pump finishes delivering the bolus volume and enters interval pause.</li> <li>When the pump is paused between bolus, the amount of time left in hours and minutes until the pump will start delivering the next bolus.</li> </ol>
	BOLUS SET	The bolus volume in mL that has been programmed.
	INTERVAL SET	The interval that has been programmed in hours and minutes. An interval is the time from the start of one bolus to the start of the next bolus.
		Exit to the main screen.

*Note: If no button is pressed within 10 seconds, the info screens are exited.*

## 12. User Settings

### 12.1 Alarm level, Backlight, Autodim, End of Nutritional Therapy Alarm

At any time, press  to review and / or adjust the pump settings. Press  again to scroll between *BEEP LEVEL*, *BACKLIGHT*, *AUTODIM?* and *ALARM AT END OF THERAPY?*. When the last setting is reached, pressing  exits the option setting mode.

Use  and  to adjust any given setting. A setting is saved in memory until it is changed. Press  to set the on-screen value to default.

Button	Device Display	Function
  	<i>BEEP LEVEL</i>	Set the audio alarm volume level from 1-3 (3 = highest). Default=2.
	<i>BACKLIGHT</i>	Set the luminosity of the screen backlight from 1-3 (3 = highest). Default=2.
	<i>AUTODIM?</i>	Set whether the screen backlight automatically dims when pump is connected to the AC power. <i>YES</i> : Screen dims after a 10-second delay. <i>NO</i> : Screen stays on permanently (not available when running on battery power). <i>Note</i> : When the backlight is dimmed, all on screen information is shown in black.
	<i>ALARM AT END OF THERAPY?</i>	Set whether the pump will warn the user the nutritional therapy is finished or not. <i>YES</i> : Pump will warn user (Default). <i>NO</i> : Pump will remain silent at end of nutritional therapy.
		Exit to main screen

*Note: If no button is pressed within 10 seconds, the settings screens are exited.*

## 12.2 Changing the Language

Step	Button	Direction
1	 2"	Press and hold  for 2 seconds (2") to activate language selection.
2		Use   to scroll to the desired language.
3	 2"	Press and hold  again for 2 seconds (2") to select the language.
4	 2"	Immediately turn the pump off to save chosen language. Restart the pump. <i>Note: If the pump is not immediately turned off, the language change will not be saved.</i>

## 13. Locking and Unlocking the Keypad

Use the keypad lock to avoid unintentional button presses, such as may occur when used in a backpack.

### Lock the keypad buttons

At any time, press and hold  and  simultaneously to lock the keypad. The pump displays a `KEYS LOCKED` message and a padlock icon appears on the screen as shown.

### Unlock the keypad buttons

When keypad is locked, press and hold  and  simultaneously to unlock the keypad buttons. The pump displays a `KEYS UNLOCKED` message and the padlock icon disappears.

#### Notes:

- When keypad buttons are locked and a button is pressed, the `KEYS LOCKED` message is displayed.
-  is always active, even when the keypad is locked.
- An alarm state unlocks the keypad.

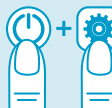
## 14. Advanced Settings

 **Precaution:** The following settings should only be adjusted by a qualified healthcare professional.

When the pump is off, press and hold  and only after the "HOLD FOR ON" message appears on the screen, press and hold  to access and then adjust the advanced settings.

Press  to scroll through the advanced settings. When the last setting is reached, pressing  cycles back to the first option. Use  and  to adjust a given setting. A setting is saved in memory until it is changed. Pressing  sets the on-screen value to default.

Step	Button	Device Display	Function
Ensure the "HOLD FOR ON" screen is displayed before pressing and holding "SETTINGS".			
1		LOCK THERAPY? NO YES	Set the previously programmed nutritional therapy as the default and prevents any changes to this nutritional therapy by the user in normal use. Use   to set: <i>NO</i> : Unlocks nutritional therapy parameters. (Default) <i>YES</i> : Locks nutritional therapy parameters.
2		ALLOW INTERVAL? YES NO	Enable /disable the use of the interval mode. Use   to set: <i>YES</i> : Interval Mode is available to the user <i>NO</i> : Interval Mode is NOT available to the user
		COMMUNICATION OFF NURSE CALL PDMS	Enable communication for either the nurse call or the Patient Data Monitoring System (PDMS) function: Use   to set: <i>OFF</i> : No communication <i>NURSE CALL</i> : Nurse call function enabled (PDMS is disabled, even if PDMS cable is connected). <i>PDMS</i> : PDMS function enabled (Nurse call is disabled, even if Nurse call cable is connected). Nurse call and PDMS cables are available as accessories (see "18. Accessories"). Contact your local Nestlé Health Science representative for details.
		SET YEAR	Use   to set Year
		SET MONTH	Use   to set Month

2		SET DAY	Use <b>+</b> <b>-</b> to set Day
		SET HOUR	Use <b>+</b> <b>-</b> to set Hour
		SET MINUTES	Use <b>+</b> <b>-</b> to set Minutes
		REV	Show the pump software version.
		SN	Show the pump serial number.
		LOCK THERAPY?	Return to the <i>LOCK THERAPY</i> screen
3			Switch off to save settings.

## 15. Alarms & Troubleshooting

### 15.1 Alarm Handling

 **Warning:** Always ensure appropriate, timely and regular patient monitoring, especially if an interruption of the tube feeding delivery may lead to a patient hazard.

When an alarm sounds:

Step	Button	Direction
1		Press the  key to acknowledge an alarm.
2		Resolve each alarm according to the specific instructions below.
3		Press the  key to resume feeding.

## 15.2 Alarm Overview and Troubleshooting

Alarm Message	Alarm Cause	Solution
	<p>The programmed total volume has been completely given.</p> <hr/> <p>The target volume programmed is less than or equal to than the <i>VOLUME GIVEN THIS NUTRITIONAL THERAPY</i>.</p>	<p>End administration or program a new nutritional therapy.</p> <hr/> <p>Clear <i>VOLUME GIVEN THIS THERAPY</i> to start a new nutritional therapy. The reprogrammed target volume must be higher than the <i>VOLUME GIVEN THIS THERAPY</i>.  <i>Note: If not cleared before each new nutritional therapy, the end of therapy alarm will sound prematurely.</i></p>
	<p>The pump door is open or the door mechanism is damaged.</p>	<p>Close the pump door.  If damaged, contact your local Nestlé Health Science representative or Nestlé approved service provider.</p>
	<p>Administration set inserted incorrectly or not at all.</p>	<p>Open the door and check for the correct installation of the cassette and tubing. Refer to page 2 and section 6.4 of the user manual.</p>
	<p>Feed container and administration set are empty down to the pump. Administration set not adequately filled.</p>	<p>If nutritional therapy is finished, end the administration, switch pump off and flush feeding tube.  If nutritional therapy is not finished:  Replace or refill tube feeding container, prime the administration set and restart feeding or,  Check for and release the occlusion, if any, between the container and the pump and restart feeding or,  Ensure the administration set is completely filled and prime if necessary and restart feeding.   <b>Precaution:</b> Ensure that the administration set is not connected to a patient when priming.</p>

 NO FLOW IN	Administration set is blocked or kinked between the container and the pump.	Check for any kinking or blockage in the administration set tubing between the tube feed container and the pump. If necessary, reposition the administration set, pump or nutrition container to ensure feed can flow freely. Open the door, check the administration set installation, and re-close the door. Restart feeding.
 NO FLOW OUT	Administration set or feeding tube is blocked or kinked after the pump, at the patient side.	Check for any kinking or blockage in the administration set tubing between the pump and the feeding tube. If necessary, reposition the administration set or pump to ensure feed can flow freely. Check that the feeding tube is clear and not kinked. Flush feeding tube if necessary. Open the door, check the administration set installation, and re-close the door. Restart feeding.
 LOW BATTERY	The battery is discharging and is running low.	
 LOW BATTERY	The battery is almost fully discharged and should be plugged in immediately; otherwise Compat Ella® will shut down shortly.	Connect pump AC power via pump AC adapter (see page 4). Charge battery.  Acknowledge the <i>READY TO USE</i> message by pressing  .
 CHARGE BATTERY	Battery is completely depleted and must charge briefly before pump can be used. Pump will display <i>READY TO USE</i> when the pump can be restarted.	
 PUMP WAITING	The pump is in pause mode and 2 minutes have passed without any pump activity.	Resume administration or switch pump off. Extend inactivity timeout by another 2 minutes by pressing  .

 REMOVE MAGNET	There is magnetic interference from an outside source.	Remove source of magnetic interference or move pump to a different location.
 ERXX SYSTEM FAIL	Internal equipment fault (system fault). "XX" = 01 – 99	Turn pump off and restart to clear the error. If the error returns or repeats, do not use the pump. Contact your local Nestlé Health Science representative or Nestlé approved service provider.

## 16. Cleaning & Disinfection

### Cleaning

The pump should be cleaned on a regular basis as follows:

1. Disconnect the pump from the AC power and turn the power off before cleaning to avoid any electric shock hazard.
2. Using a damp sponge or cloth, clean the pump with warm, soapy water.
3. The pump is water resistant and may be placed under warm running water for cleaning.
4. Dry carefully with a soft cloth before reusing.

*Note: To avoid damaging the pump do not use sharp objects or aggressive cleaning techniques in the pressure sensor area (see pump and accessory overview items G and H)*

### Disinfection

To determine proper disinfection procedure in a particular institution, follow your local institution policy. It should be noted that some harsh disinfectants may damage or discolor the pump housing which may void the warranty in this regard. Upon request, Nestlé Health Science will provide a list of approved disinfectants.

### Warnings!

- Do not submerge the pump in water.
- Do not use in the dishwasher.
- To avoid electrical shock, never clean while the pump is on or plugged in.
- Do not autoclave.
- The use of harsh cleaners/disinfectants and isopropyl alcohol, may damage the pump housing.

## 17. Maintenance

-  **Warning:** If the pump is damaged or altered, do not use it. Contact your local Nestlé Health Science representative or Nestlé approved service provider.
- The Compat Ella® portable enteral feeding pump has no user-serviceable parts. No calibration is required. The Expected Service Life (ESL) is 10 years. The required maintenance interval for the Compat Ella® pump is every 5 years or sooner if the battery no longer holds a charge. In the case of a hardware or software malfunction, contact your local Nestlé Health Science representative or Nestlé approved service provider to repair or replace the pump.

## 18. Accessories

### Patient Data Management System (PDMS)

A separate cable is available as an accessory to enable connection of the pump to an approved PDMS system. When connected to an approved PDMS system, and PDMS is enabled, the pump will allow tracking of key information such as Rate and Volume Given via the PDMS. See advanced settings to see how to enable PDMS. Contact your local Nestlé Health Science representative for details.

### Nurse Call

A separate cable is available as an accessory to enable connection of the pump to a nurse call system. When connected to an approved nurse call system, and nurse call is enabled, the pump will send an alarm signal to the nurse call management system. See advanced settings to see how to enable Nurse call. Contact your local Nestlé Health Science representative for details.

### Backpacks

Adult and Junior backpacks are available as accessories. Contact your local Nestlé Health Science representative for details.

- Connect tube feeding container, administration set and pump and fully prime the system before placing into the backpack.
- Secure the pump in the backpack. Avoid excess personal items in backpack as they may cause kinks in the tubing and interfere with pump operation.
- Make sure the tube feeding container is hooked securely at the top with the set connector at the bottom.
- Make sure the administration set and tubing are free from kinks, which could cause a blockage.
- Use extra care to avoid introducing air into the system.
- Maintain backpacks in an essentially upright position to keep tube feeding container higher than pump.
- When using an open container or bag, eliminate extra air in the system as this will help avoid Air-in-Line alarm.

### Table Stand / Backpack Insert

A table stand is available as an accessory. It can also be used with a backpack. Contact your local Nestlé Health Science representative for details.

*Note: some accessories might not be available locally.*

## 19. Limited Warranty

All service should be performed by authorized personnel. Unauthorized repairs will void the warranty.

Pump, pole clamp and AC power adapter are guaranteed by the manufacturer for a period of 24 months from date of delivery.

- The warranty covers repair and replacement of components proven to be defective in material or workmanship.
- The warranty does not extend to units that have been altered or repaired by unauthorized persons and malfunctions due to improper handling and wear.
- The manufacturer can only be held responsible for the safety, reliability and performance of the feeding pump if:
  - a. Assembly, extensions, readjustments, modifications or repair have been performed by persons authorised by the manufacturer,
  - b. The electrical installation of the site where the pump is used complies with the requirements of the IEC regulations, the pump is used as specified in the Operating Instructions,
  - c. The pump is operated with the administration sets specified by the manufacturer.

## 20. List of Symbols

	Batch code		Type BF applied part		Humidity limit (storage)
	Catalogue number		Class II equipment		Atmospheric pressure limit (storage)
	Serial number		Keep away from rain		Temperature limit (storage)
	Globe code		Keep away from sunlight		Battery
	Direct current		Recyclable li-ion battery		Date of manufacture
	Input voltage		Mechanical and fluid ingress		Fragile, handle with care
	Manufacturer		Waste of electronic and electronic equipment		This side up
	Package content		No open flame: Fire open ignition source and smoking prohibited		Warnings and precautions
	Recyclable material		Not for IV use		CE mark and Notified Body
	Green dot				
	Follow instructions for use				

## 21. Specifications

### Mechanical

- Dimensions: L:140 mm x B:140 mm x H:62 mm.
- Weight: 499 g

### Operational

- Flow Rate: 1 mL/hr to 600 mL/hr
- Volume: 1 mL to 4000 mL
- Increments: 1 mL
- Feed Until Empty (Continuous) CONT
- Priming Rate:  $\geq 700$  mL/hr
- Accuracy:  $\pm 5\%$  according to the following parameters:
  - Rate: 100 mL/hr
  - Volume: 100 mL
  - Liquid: Water
  - Administration Set: New, previously unused set
  - Up to 1500 mL delivered volume\*
  - \*Delivery of more than 1500 mL with one set may affect accuracy.
- Head Height above rotor: 30cm
- Head Height Effects on Accuracy:  $\leq 2.5\%$  per 30cm of head height.
  - Head height range: -30cm to 60cm
- Back Pressure Effects:  $\leq 0.5\%$  per PSI
- Occlusion Detection Pressure:
  - Upstream: -34 kPa for 5 seconds
  - Downstream: 83 kPa for 30 seconds
  - 103 kPa for 5 seconds

*Note: Low flow rates may significantly extend the time to occlusion alarm*

- Air Detection 1.5 mL
- Operating Temperature:  $+5^\circ$  to  $+40^\circ$  C
- Operating Humidity: 15% to 93%, non-condensing
- Operating Atmospheric Pressure: 65 to 106 kPa

### Storage

- Storage Temperature:  $-20^\circ$ C to  $+60^\circ$ C
- Storage Humidity: 10% to 95%, non-condensing
- Atmospheric Pressure: 50 to 106 kPa

### Power

- Voltage:
  - Input: 100-240 V; 50-60 Hz;
  - Output: 5 VDC 2A
- Fuse Type: Single blow fast acting
- Battery Type: Lithium Ion
- Battery Charge Time:  $\leq 6$  hours
- Battery Life: 24 hours, at 125mL/hr

### Battery capacity decay due to:

**Storage:** For optimal battery life, do not expose battery to elevated storage temperatures for extended periods. Store below  $50^\circ$ C when possible and do not exceed  $60^\circ$ C storage temperature. For extended storage, store battery at low temperature and low charge state.

**Aging:** Please note that the battery capacity numbers noted below are estimates and that actual battery capacity may vary significantly depending on factors such as actual use, charge cycles and temperature.

Pump Age	Approx Run Time at 125 mL/Hr
1 yr	24 hrs
2 yr	24 hrs
3 yr	20 hrs
4 yr	16 hrs
5 yr	12 hrs
Above 5 years	Service Pump*

\*Service interval for the Compat Ella® pump is every 5 years. Battery will be replaced.

In the case of a malfunction, contact your Nestlé Health Science representative or Nestlé approved service provider to arrange for the pump to be repaired or replaced.

- Electric shock protection: Class II
- Mechanical and Fluid ingress: IP25

The Compat Ella® is designed in accordance with EN 60601-1-2 ed.3. standard for electromagnetic emissions and immunity.

## 22. Recycling and Disposal

### **Disposal of the pump:**

Follow your local laws, directives, standards and guidelines for the disposal.

### **Disposal of the battery:**

The battery should be removed as part of the disposal of the pump and processed as per specific local requirement.

### **Disposal of the packaging material:**

All packaging materials have been selected according to environmentally compatible and disposal aspects and can be recycled. Please send old packaging materials to the relevant collection and reprocessing system. This way, you will contribute to the recycling or raw materials and the reduction of waste.

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Nestlé Italiana S.p.A.
Nestlé Sverige AB